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*The implant is a part of the ***Sky-Implant-System*** and should only be used with its original components and instruments according to the instructions and recommendations of the Star-Group-International p.l.c..*

Further product information can be found in the main catalogue or it may be downloaded by visiting our website at. PDF file format documents in other languages can be downloaded from the website.

**STAR-GROUP-INTERNATIONAL
IMPLANT-DEVELOPMENT
& TECHNOLOGY GMBH**

Hallplatz 2
90402 Nürnberg GERMANY
Tel: +49 (0) 911-23 600 90
Fax: +49 (0) 911-23 600 97
www.star-group-international.com
REV. 03/2002

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*****SKY-IMPLANT-SYSTEM*****

*The ***Sky-Implant-System*** is a cylindrical self-tapping dual thread dental implant, which is available in various diameters and lengths. It can be used in a one-stage or two-stage surgical procedure in the upper or lower jaw wherever a cylinder-shaped implant is indicated.*

IMPORTANT INFORMATION

This implant-system should only be used by experienced dentists or surgeons who either have passed examinations in implantology or have obtained a certificate at the IFZI (International Education Centre for Dental Implantology) in Nuernberg or any authorized institute by Star-Group-International.

*The appropriate use of the ***Sky-Implant-System*** exclusively relies on the certified user; therefore, all liabilities for damages caused hereby are excluded. It is imperative to follow the fundamental guidelines which require aseptic surgical procedures.*

The Star-Group-International guarantees the sterility of the implants by use of gamma irradiation if stored correctly and packaging is kept in intact conditions until the expiration date. Resterilization of the implants is not recommended.

Please keep in mind that all the drills are for "single-patient" use only. Instruments must be cooled sufficiently with sterile sodium chloride solution during operation.

During preimplantological planning all local and systemic contraindications should be taken into consideration.

DOCUMENTATION


Every implant's package includes adhesive labels with all dates that are relevant for documentation. They can be attached to the patient's chart and to the implant-pass.

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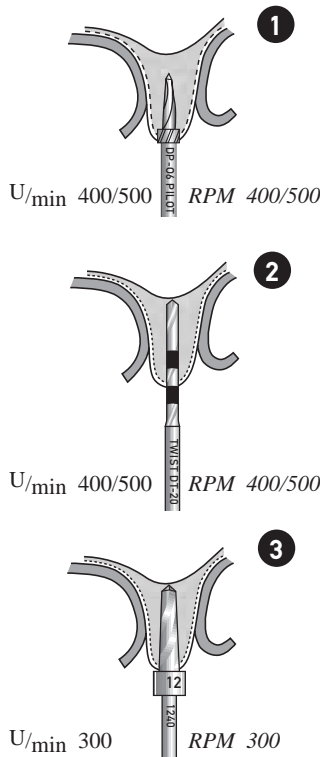


INSTRUCTIONS

➊ After gentle deperiostation and preparation of the alveolar ridge with a round burr, a pilot drill is used to guide the location and direction of the implant without reaching the planned depth. Pilot drills are available in lengths of 6 and 12 mm. The 6 mm version of the pilot drill can also be used to extend the cortical bone up to 3 mm in diameter.

➋ A twist drill of 2 mm in diameter will reach the final depth with increments of 8, 10, 12, 14 and 16 mm according to the length of the implant that is being planned.

➌ The final drill will extend the implant socket to a diameter of 3.5 mm, where a stop is used to avoid further sinking of the drill. If the alveolar ridge is wide enough to use an implant of wider diameter, implant drills with a diameter of 4.0 or 4.5 are available. Depending on bone quality the use of the cortical finisher may be indicated.

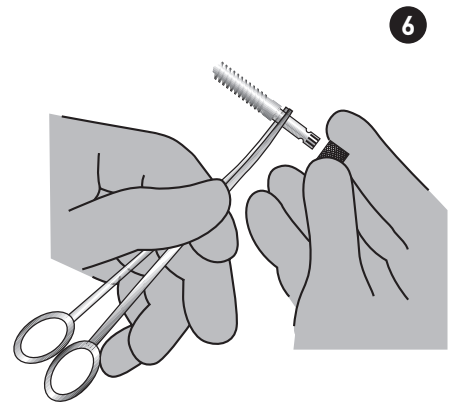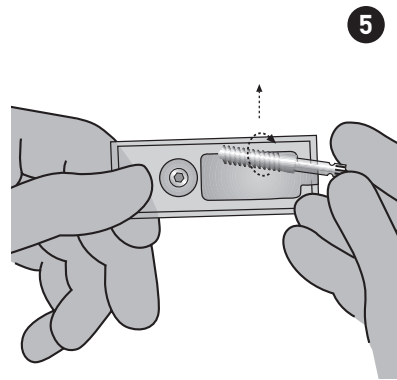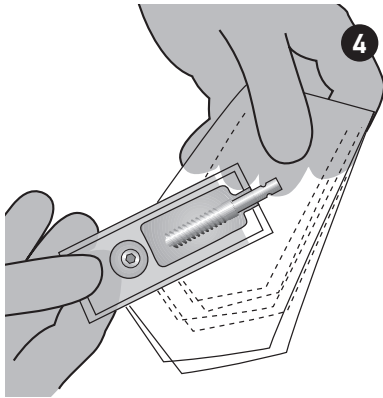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STAR-GROUP-INTERNATIONAL
IMPLANT-DEVELOPMENT & TECHNOLOGY GMBH

*Prior to inserting the implant, the implant socket has to be rinsed thoroughly. Now the original packaging will be opened **4** and the implant will be removed from the box **5**. The universal forceps serves as a retainer to place on the small finger key (Star-Driver) **6**.*

***7** The implants can now be inserted with the finger key (Star-Driver) into the implant socket.*

***8** Using a ratchet which is placed over the finger key (Star-Driver), the implant can be carefully inserted to the correct implant depth without overheating the bone. (No instrument changes are necessary). Finally an impression is directly taken using the implant carrier resp./impression post.*

Important notice:




The implant carrier should be replaced with the solid ratched (SD-24) in D1- and D2-bone for the definitive insertion.

***9** Following this procedure the impression post/implant carrier can be removed and replaced by a healing screw (two-stage procedure) or gingiva former (one-stage procedure).*

After about a 12 week submerged healing period in the mandible and 20 weeks in the maxilla, the one-stage implants are ready for second-stage surgery (uncovery phase). For this reason, the gingivaformer is placed in adequate length.

Reproduction or publication in part or as a compute document can only be done by permission of

STAR-GROUP-INTERNATIONAL
IMPLANT-DEVELOPMENT & TECHNOLOGY GMBH

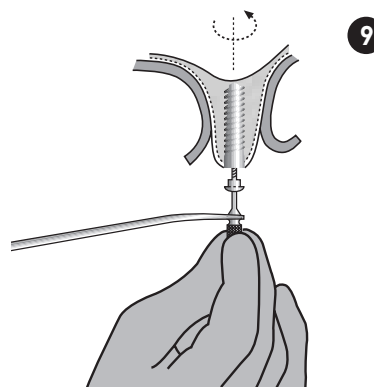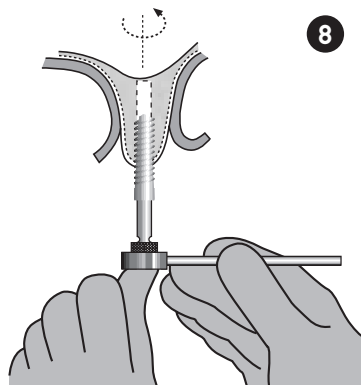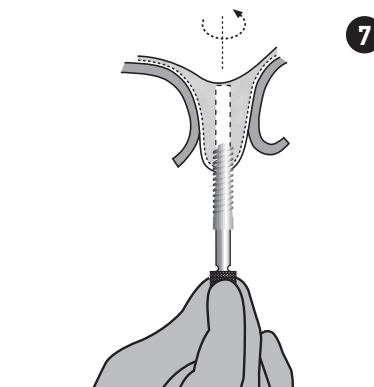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Sterilized Product, electromagnetic or ionic radiation sterilisation process

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See directions for use

Verfallsdatum

Expiry date

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One use only

REF 1234

Artikelnummer

Article reference

LOT 1234567

LOT-Nummer

Batchnumber

CE 0482